



SAMARIT

improving patient care

SAMARIT "no lift" Transfer Systems

NEW

Rollbord™

HIGHTEC

For
HEAVY
use!

The safe way to transfer patients

A legal requirement in some countries, taken for granted in others – working conditions should be as comfortable as possible. Healthcare workers should avoid lifting when transferring patients. **SAMARIT Products** assist with transfer, without the need to lift. The products are designed to facilitate the transfer without using a lot of effort, thus making the work easier for the carers and kinder to their backs.

STANDARD

foldable



Suitable for radiology, operating theatre, intensive care units and all special wards.

Size: 177 x 50 cm / 87 x 50 cm, 2.7 kg
70 x 20 inch / 35 x 20 inch

Anti-bacteriologic, anti-fungic, flame retardant, security endings, skin friendly coating and easy to clean.

Order No. 440.0100

ICU-BARIATRIC

non-foldable



Suitable for Intensive Care (ICU) nursing services, on air and decubitus mattresses.

Size: 177 x 50 cm, 3.8 kg
70 x 20 inch

Anti-bacteriologic, anti-fungic, flame retardant, security endings, skin friendly coating and easy to clean.

Order No. 440.0200

MINI-GYN

non-foldable



Suitable for urology operating theatres, gynaecology operating theatres, wake-up stations and emergencies.

Size: 91 x 50 cm, 1.4 kg
36 x 20 inch

Anti-bacteriologic, anti-fungic, flame retardant, security endings, skin friendly coating and easy to clean.

Order No. 440.0300

5 different models to suit every hospital application

- High performance
- Radioluscent
- Non-toxic materials
- Anti-bacterially surface
- Safe and comfortable for patients
- Prevents back injuries to care workers
- Skin-friendly (no latex)
- Easy to clean and disinfect – hygiene approved
- Disposable covers available
- Tear resistant nylon cloth
- Bridges gaps up to 20 cm
- Lightweight, easy suspendable

ROLLMAT - Special OEM Product

foldable

Ask for special offer!



Specially made for radiology and MRI scanners.

Size: 180 x 50 cm, 3.8 kg
71 x 20 inch



Order No. 440.0700

SURGIBOARD *non-foldable*



Suitable for operating theatre, X-Ray, various wards (replaces aluminium rollers).

Size: 126 x 40 cm, 1.5 kg
50 x 16 inch

Anti-bacteriologic, anti-fungic, flame retardant, security endings, skin friendly coating and easy to clean.

Order No. 440.0400

MRI/CT *foldable*



(Option)

Suitable for transfer of injured patients on/ from stretcher.

Size: 177 x 40 cm, 2.5 kg
70 x 16 inch

Anti-bacteriologic, anti-fungic, flame retardant, security endings, skin friendly coating and easy to clean.

Order No. 440.0500

DISPOSABLE COVER *Suitable for all Rollbord models*



To protect Rollbords from blood and dirt. **Size:**

200 cm / 135 cm / Ø 60 (tube)

200 cm / 79 inch **Order No. 440.1000**

135 cm / 53 inch **Order No. 440.1400**

SAMARIT "no lift" Transfer Systems make day-to-day work with patients easier

SAMARIT also gives comfort and security to patients

In hospitals, where patients may need to be transferred carefully several times a day, **SAMARIT** makes life easier for the carer and more comfortable for patients.

SAMARIT Transfer Systems – a great help to healthcare professionals – worldwide!

If you imagine how many transfers are carried out every day in hospitals and within communities then you will appreciate the true value of the original **SAMARIT** systems. In fact, over 150'000 **SAMARIT** Transfer Systems are being used by nursing professionals working worldwide.

All of them find that their work is made easier when using **SAMARIT** products.

different model weighs 1.4 - 3 kg and
can be set up quickly – anywhere



The safe and secure way to transfer patients



Turn the patient onto the side, slide the board under the body and return the patient to the original position.



Gently push the patient at the shoulder and hip, without applying force, and transfer.



Turn the patient onto the side. Remove the Rollbord starting at the feet.

Environmentally friendly material* SAMARIT transfer aids are made of ecologically safe materials. They can be disposed of in an environmentally friendly manner and if a fire should occur they do not produce toxic fumes.

* Teflonised sliding fabric around a polyethylene foam core, coated polyester cloth, PE security endings.

High quality standards
SAMARIT transfer aids have:

- CE mark
- Patents and international design patents

Easy cleaning

The Rollbord can be cleaned best with *SamaClean Wipes* or any hospital disinfectant. For anticipated soiled conditions use the Rollbord disposable cover. It covers ends of Rollbord and allows any residue to be folded untouched into the cover, leaving a clean surface. Repairs can only be carried out by authorized personnel (warranty).

www.samarit.com

Ask for separate brochures for the other products in the SAMARIT range!



SAMARIT Rollbord ECOLite



SAMARIT Rollbord H-Line



SAMARIT Transglide



SAMARIT Nursing Aids

Download your language from our website or ask your local distributor.



Samarit Medical Industries Inc.
55 Fleming Drive Unit #9, Cambridge
Ontario N1T 2A9, Canada
Phone: 888 658 2206 Fax: 519 658 2209
e-mail: info@samarit.ca - web: www.samarit.ca